


The Grid 2: Input Settings – General

The Grid 2 is designed to make controlling both the Grid 2 and Windows computers as easy as possible for people who find a standard mouse, keyboard and/or touch screen difficult to use.


The access methods for the Grid 2 are found in the user settings for each grid set. These settings are specific to a particular grid set. This allows people to set up multiple copies of the one grid set with different access methods or if more than one person uses a computer they can each have their own grid set with their specific access needs saved.

There are 2 ways to enter the user settings for a grid set.


1. From the Grid Explorer window highlight the grid set and then from the left hand task bar select **user settings**.
2. From within the grid set itself, select **edit** → **user settings**

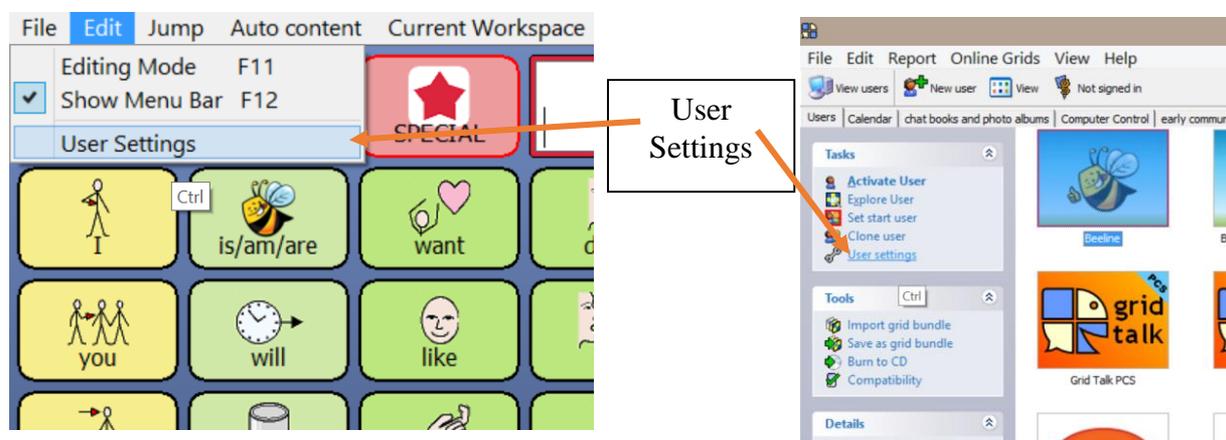


Once you have opened the User Settings dialogue box you will notice a heading on the left hand side entitled Input Settings. Select this to see the various input settings available.

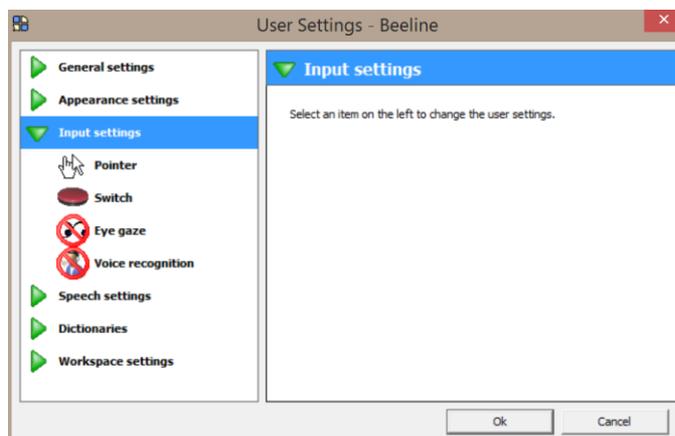

For each input method there are a number of options that allow you to customise the use of that method. For example if you select the **Pointer** option you will see headings to select the various pointer settings available.

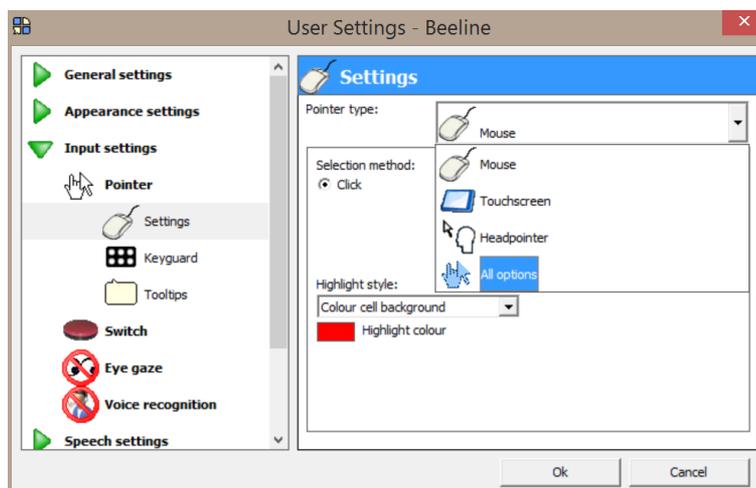

Alternatively if you select **Switch** you will see various options related to configuring your computer or communication device to use switch access and also related settings such as scanning pattern and speed.

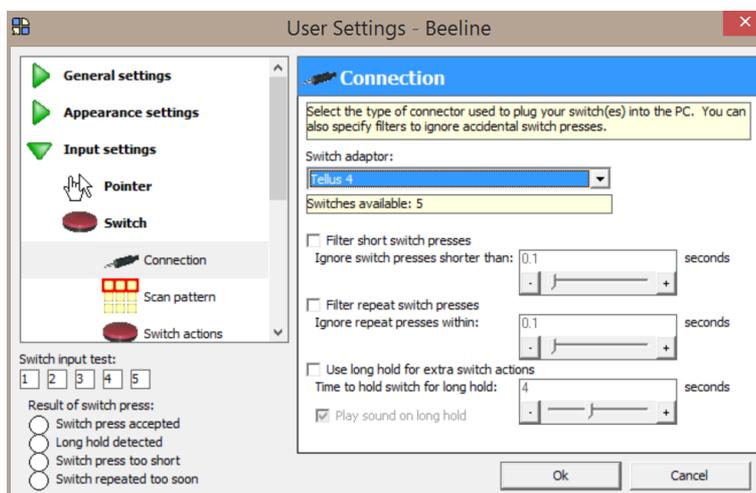

For further information regarding user and input settings please refer to the additional information sheets on our website.